

G-DRESS / G-DRESS SWIM PROOF/G-BAND SPORTY Instructions for use

- First apply the ointment, cream or any antiseptic recommended by your doctor.
- Peel open the pouch and take out the dressing with clean hands.
- Peel off one side of the release liner by pulling it from central edge.
- Apply the dressing by the side, which is free from the liner, keeping the pad on the wound area. Simultaneously hold the other liner and keep on pulling the liner and applying the dressing.
- Do not touch the pad with bare hands.
- ***WARNING: Some persons may be sensitive to the adhesive, if any allergic reactions appear, discontinue the use and consult doctor.***